

CCV/CPV Ag

Canine Coronavirus/Parvovirus Antigen Combo

Principle

Canine CCV/CPV Ag Combo is a chromatographic immunoassay kit for rapid and qualitative detection of CCV and CPV antigen in canine feces.

The nitrocellulose membrane of the kit is immobilized with CCV-specific antibody in line 2 and CPV-specific antibody in line 1, and with anti-mouse antibody in control line. And also, CCV-specific antibody and CPV-specific antibody is conjugated to the colloidal gold particles respectively. This conjugate is placed on a polyester or glass fiber as conjugate pad.

When the sample is dropped into the sample well on the device, the solubilized conjugates migrates with the sample by passive diffusion and both the conjugate and sample come into contact with the antibody that immobilized onto the nitrocellulose. If the sample contains CCV antigen, the result is visible as red line within ~10 minutes in the line 1 and contains CPV antigen, the result is visible as red line 2 on the membrane. The solution continues to migrate to encounter a control reagent that binds a control conjugate, thereby producing another red control line.

Materials Provided

Canine CCV/CPV Ag Combo (Art.No. 40401-05/06) contains the following components:

1. 5/10 test devices individually foil-pouched with a desiccant.
2. 5/10 specimen tubes containing dilution buffer
3. 5/10 sterilized swabs
4. 5/10 disposable droppers
5. One (1) instruction manual for use.

Precautions

1. The presence of humidity may decrease the stability of the reagents.
2. Do not use the kit after the expiration date and do not freeze.
3. For veterinary use only.
4. Wear protective gloves while handling samples and wash hands thoroughly after test.
5. Dispose all the specimens and kits properly after test, in accordance with GLP.
6. Never use reagents from another kit.
7. Discard the Assay solution if it is contaminated with bacteria or mold.

Storage & Expiration

1. Canine CCV/CPV Ag Combo should be stored between 2 to 30°C (36~86°F)
2. Expiration date of this kit is 18 months after its manufacture date.


Specimen Collection and Storage

Specimen of this is canine fecal material (feces). And, it should be obtained and handled by standard methods for sample collection. The feces specimens must be tested as soon as they are prepared. If necessary, they may be stored at 2~8°C for up to 48 hours. If longer storage is required, the sample may be frozen at -20°C.

Test Procedure

1. Place all specimens, test devices and allow them to room temperature prior to testing (15~30min).
2. Remove the test device from the sealed pouch, and place it on a clean and plain place.
3. Collect the feces specimens at 6 different points or directly take the specimens from the canine anus.
4. Insert the swab into specimen tube containing dilution buffer.
5. Mix the swab specimen with dilution buffer.
6. Add 4 drops (approx. 120 µl) of mixed sample by the disposable dropper into the sample well.
7. Interpret the testing result within 10 minutes. Do not read the result after 20 minutes.

TEST PROCEDURE:

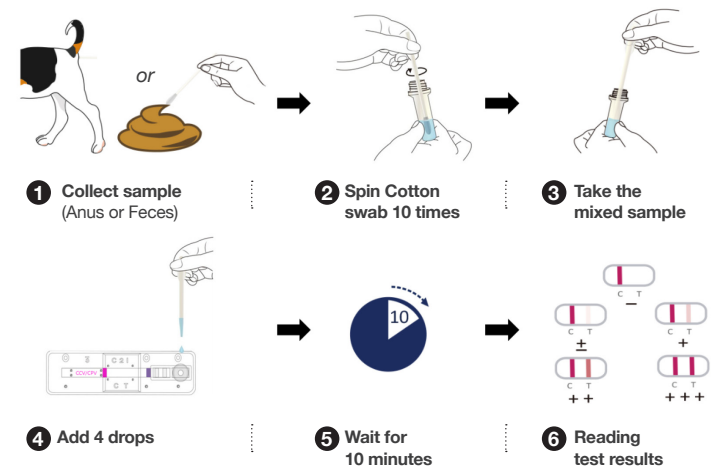

Interpretation of the Test

1. A color band will appear in the left section of the result window to show that the test is working properly. This band is the Control band (C).
2. The center and right section of the result window indicates the test results. If another color band appears in the right section of the result window, this band is the Test band (1, 2).

A. Negative result


The presence of only one band in C band within the result window indicates a negative result.

B. Positive result

- 1) The presence of C, 2 band indicates a CCV Ag positive result.
- 2) The presence of C, 1 band indicates a CPV Ag positive result.
- 3) The presence of C, 2 and 1 band indicates a CCV and CPV Ag positive result.

C. Invalid result

If the control band is not visible within the result window after performing the test, the result is considered invalid. The directions may not have been followed correctly or the test may have deteriorated. It is recommended that the specimen be re-tested

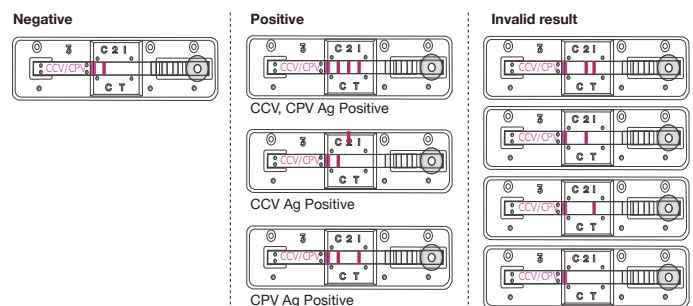

Limitation of the Test

Canine CCV/CPV Ag Combo is designed for primary screening test of canine coronavirus and parvovirus infection. This kit can provide fast and easy way to get a very accurate result, but do not completely exclude the possibility of false positive or false negative result caused by various factors. So, refer to the result of this kit, please make a final decision with clinical manifestation, other test result, and veterinarian's view, collectively.

Manufactured by Zet Biotech Inc for LifeAssays AB